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## INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

### FUNCTION

The silicone pads **1** integrated into the bandage gently massage out the swelling on the side of the Achilles tendon during movement and stimulation circulation. The openly attached heel wedge made of vibration reducing material (no ‘floating effect’ when walking) can be employed if, in the opinion of your doctor, a temporary reduction of strain on the Achilles tendon is indicated. The extra heel wedge for the healthy side prevents a Beckenschiefstand.

### INDICATIONS

#### Conservative treatment:

- Achillodynia (pain in the Achilles tendon)
- Tendinitis / Peritendinosis of the tendon
- Paratenonitis
- Sub-Achilles bursitis

#### Postoperative treatment:

- Operations on the gliding tissue of the Achilles tendon
- Sub-Achilles bursitis operations
- Achilles tendon rupture

### CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

### SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

### FITTING

– Pull the bandage over the foot.

- Properly positioned, the pressure pads should be located in the hollows on either side of the Achilles tendon. The pockets for these pressure pads are purposely kept slightly larger, so that the pressure pads can to some extent be moved to the ideal position in the hollows.
- The ACHILLODYN® should not fit too tightly. If it exerts an uncomfortable amount of pressure, choose one size larger.
- Both heel wedges should be positioned at the very back of the shoe.

### IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper

use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

### QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

### CARE

Wash ACHILLODYN® in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. The enclosed heel pad can be cleaned with a moist cloth.

**TIP:** Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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