E

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GB


Immobilisation is a basic principle for treatment of injuries and overstrains, as recovery process will be considerably accelerated and consequential damages prevented.

Shoulder Arm supports are relieving affected tissue structures (joints, ligaments, capsules, cartilages).


With the BORT OmoBasic® Closed Form, you can place your shoulder joint and elbow comfortably and securely in the adducent position close to your body.

Instructions for use:


A specialist should put on the BORT OmoBasic® Closed Form for you.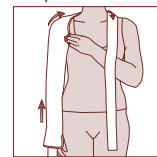


1. Place the hand on the injured arm through the tube's opening and initially pull the tube upwards until the tips of the fingers are visible. Hold the tube securely at the wrist and with your other hand pull the support upwards toward the shoulder joint. When doing so, be careful not to pull too firmly on the shoulder joint. Now run the shoulder belt around the shoulders and neck back toward the front of the body.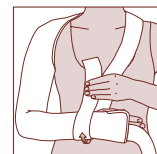

2. Bend the injured arm at the elbow at a 90° angle, unless otherwise directed. In order to stabilize the forearm, wrap the shoulder belt under the wrist and back upwards, and then fasten it using the Velcro strip.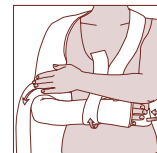

3. Run the body belt which is attached to the hand section away from the hand, around the back, and toward the arm which is being immobilized. Now run the belt forward between the body and forearm, around the upper arm, and then around the back of the body.

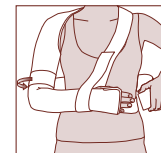4. Securely fasten the end of the belt to a section of body belt already wrapped around the body.

Indications:

Sprains, contusion, injuries to the rotator cuff, dislocation, subcapital humerus fractures, fractures of the humerus ball and shaft, after surgical intervention, condition after hemiplegia.

Contraindications:

Do not use with Artery Occlusive Disease (AOD) and stasis in the lymphatic system.

Composition:

37% Polyamide, 30% cotton, 16% Pur-Ester, 12% polyester, 5% elastodies

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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