I

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GB

Ankle Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. At the same time the support helps to a high degree to avoid wrong movements. The pad provides a massage effect which reduces swelling tendencies or which helps to an earlier restrain of swelling.

Indications:

After removal of plaster cast, sprains, arthrosis, ligament instability.

Contraindications:

Large-area skin injuries, large-area edemas, peripheral neuropathy (e.g. diabetes mellitus). Artery Occlusive Disease (AOD) and Chronic Venous Insufficiency (CVI). Oversensitivity against one of the materials used.

Composition:

57% Polyamide, 28% elastodies, 8% spandex, 7% viscose

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.
In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.
While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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