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(DE)

## PROTEKTOR der Rollatorschirm.

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**PROTEKTOR** the rollator umbrella.

## Application area

PROTEKTOR the rollator umbrella offers optimum protection against the rain and the sun, while the rollator can be led with both hands. Applicable for all 4-wheel rollators, also for wheelchairs. Not applicable for Delta-walking-wheels.

## Technical measurements:

weight: 1 kg      Ø opened umbrella: 105 cm

## Safety Instructions

This operating instruction is to read and consider before you use the rollator umbrella.


- !** Please check the constant seat of the screws continuously before using the rollator umbrella. Please attend that the umbrella doesn't affect the brake operation of the rollator. Also lightening, bell and the stick-holder are not allowed to be affected.
- !** Before the first use by rain and in the road traffic please make yourself familiar with the mechanism of the umbrella and attend the amount and height of the assembled rollator umbrella.
- !** By activating the automatic-opening at pushing the button please hold the umbrella afar from the body and attend of other persons to avoid injuries. Please also attend the required space of the rollator umbrella while opening to avoid damages.
- !** By closing the rollator umbrella please avoid to clamp your fingers.
- !** In the closed status please turn the umbrella in and close the hook-and-loop-fastening belts.
- !** After parking the rollator please always close the umbrella, turn the umbrella in and close the hook-and-loop-fastening belts.
- !** While using the rollator umbrella in the road traffic please attend on a free field of view. Don't constrain other road users.
- !** We although advise to close the umbrella if there's too much wind.
- !** The rollator umbrella it's not meant for carrying and moving the rollator.

# Operating Instructions, page 2

**PROTEKTOR** the rollator umbrella.

## Contents

1. Rollator umbrella with hinge joint
2. Fixationplunger, already pre-assembled for Ø 22 mm
3. Fixationcuffs for tube Ø 18/ 20/25 mm
4. Set of screws (2 pcs.) M6 x 45mm
5. Set of screws (2 pcs.) M6 x 75 mm
6. Allen key

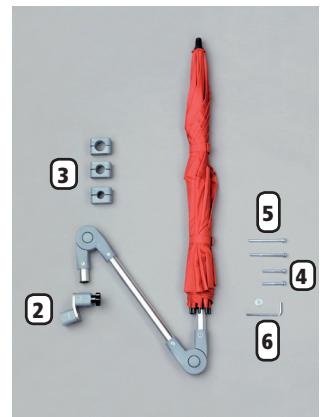## Assembly

The fixationplunger of the rollator umbrella is already pre-assembled for the fixation on the standard rollator with 22 mm roundtube (Pict. 1).

Please mount the fixationplunger on the height-adjustable slide-grip of the rollator by the help of the Allen key. The adoption-hand-wheel points to the front of the driving direction (Pict. 2).

For the fixation on the 18/20/25 mm round tube please change the accordant fixationcuffs.

For the fixation on the ovaltube please choose the right set of the fixationcuffs so that a constant seat is assured (Pict. 3+4).

For the fastening on TOPRO „TROJA“ please use a 22+18 mm fastening clip (Pict. 7).

Pict. 1

Pict. 2

Pict. 3

Pict. 4


Pict. 5


Please situate the aluminium-stab of the rollator umbrella into the fixationcuffs and turn the adoption-hand-wheel around until it's attached. The rollator umbrella is now assembled and can be expanded by the easy activity of the pressure-mechanism (pict. 5+6).

Pict. 6

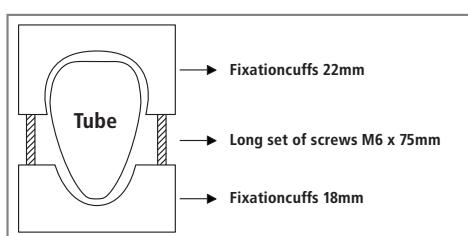Pict. 7
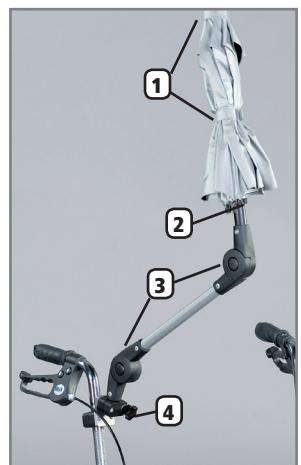
## Handling

To open the rollator umbrella, which is already assembled on the rollator, please release the hook-and-loop fastener (1) and activate the automatic-opening at the push of a button (2). Attention, the umbrella swivels out.

Through the easy pressure- mechanism on the solid hinge joint (3) you can put the rollator umbrella into different positions.

Another positioning is possible through the release of the adoption-hand-wheel (4).

Please pull tight again afterwards.

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