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drive
MEDICAL DESIGN & MANUFACTURING

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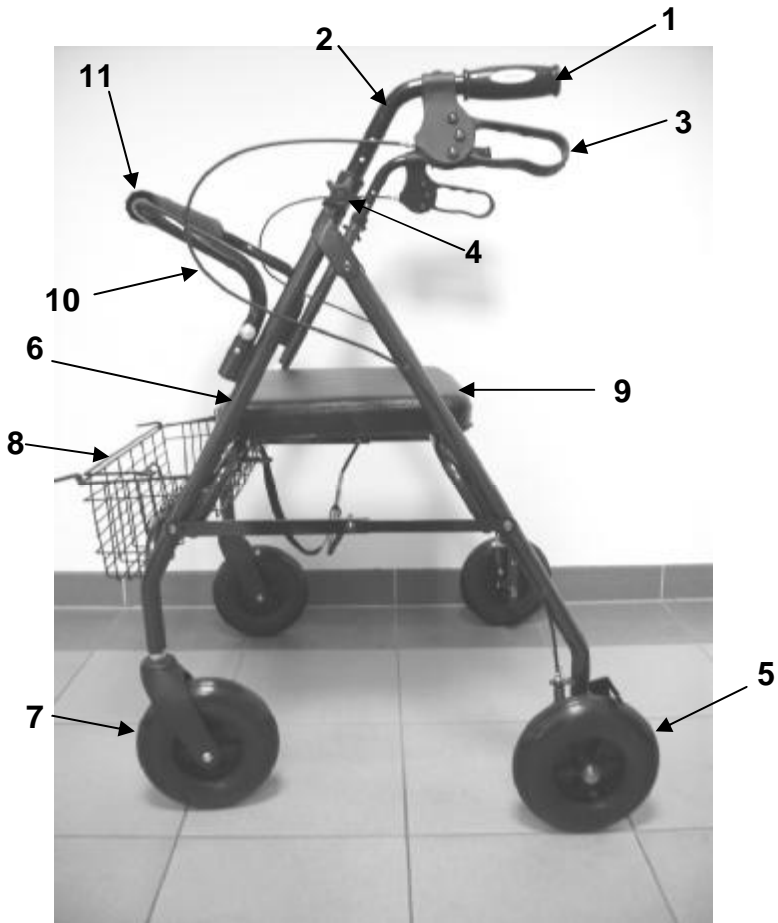
Walking device for indoor and outdoor use for compensation and improvement respectively of restricted ability of walking.

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This product is suitable for the reapplication. With the processing the standards of the manufacturer must be considered.

This standards are made available on inquiry.

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1. Walking device
2. Basket
3. Tray
4. backrest
5. Operating instructions

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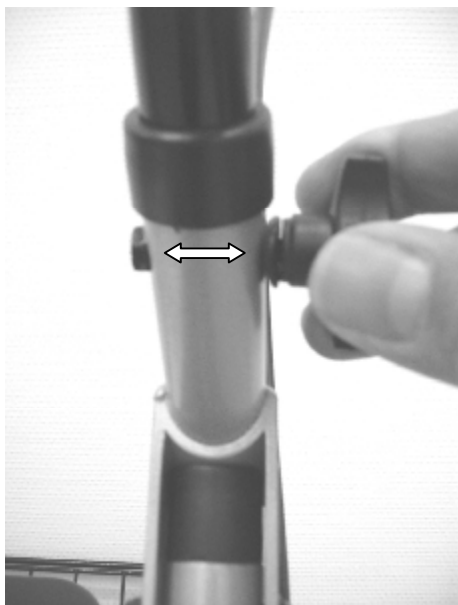
Insert the back support from the top into the reception tubes. Press more-over the locking pin.

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Insert the tube of the adjustable handgrip into the frame at least to the "first hole". Insert the locking screw through the pipe from the inside and fix with washer and star clutch screw.

To change the height setting of the adjustable handgrip the locking screw by turning off the star clutch against the clockwise solve. Then ascertain as on top described the clutch by desired height.

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*Position the walking device on its
wheels. Push the front frame forward
by pressing on a handgrip down until
the walking device stands on four
wheels. Then press the seat down.*

Korb anbringen – Fitting with basket and tray

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*Hook in the basket with the hooks (see
circle) at the frame under the seat
plate and then pivot it down until the
basket attaches the pipe.*

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For braking the walking device pull both brakes upwards to the handgrip (see picture).

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For parking the walking device press down the brakes until they click in the parking position (see picture). For unlocking the brakes pull them upwards towards the handgrips until they are unlocked (both brakes are in a middle position).

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The effect of the brakes can be affected by abrasion of the wheels. Therefore test regularly the brakes.

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For desired braking strength adjust the brake either manually at the screw on the housing of the brake or with the suitable tool (not included) on the brake.

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*Position the walking device on an even and solid surface. **Then lock the brakes (parking position).** Walking device must stand and must not slip. Sit straight up and do not lean backwards*

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To fold the Rollator pull tseat in the Rollator upwards! Then slant Rollator against the run direction, the frame wrinkles. Now the Rollator can be move like a trolley or be stowed away.

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*Use only as a walking device!
Use only on even and solid surfaces.*

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Walking device must not be used for standing on or as a wheelchair!

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*Use basket only for transport of objects and do not overload.
Max. load: 5.0 kg*

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*Clean regularly by hand with a damp cloth!
Do not use any aggressive detergents!
Screws of walking device must be checked regularly and if necessary re-tightened.*

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Steel, powder coated

Typenschild – Type label

The image shows a rectangular type label for a 'drive' Rollator Walking Aid 'GoLite 200'. The label contains the following information:

- 1** → **Rollator Walking Aid "GoLite 200"** (Product name)
- 2** → **REF** 770 100 100 (Article number)
- 3** → **SN** XXX XXX (Serial number)
- 4** → **33 / 08** (Production date)
- 5** → **i** (Information icon)
- drive** (Brand logo)
- CE** (CE mark)
- Max. Belastung - Max. load: 200 kg**
- Max. Belastung Korb – Max. load basket: 5.0 kg**
- Die Griffe müssen parallel zur Gehrichtung eingestellt sein!
Handgrips must be adjusted parallel to the walking direction!
- Für Innen- und Außenbereich – *For indoor and outdoor use!*
- Max. Breite – *Max. width: 780 mm*
- Drive Medical GmbH & Co. KG · 88316 Isny/Allgäu (Germany)

- 1 Produktname – *product name*
- 2 Artikelnummer – *article number*
- 3 Seriennummer – *serial number*
- 4 Produktionsdatum – *date of manufacturing*
- 5 Gebrauchsanweisung beachten – *Attention: See operating instructions*

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
Stand - *as per*: **08 / 2008**

We issue a warranty of 24 months from date of purchase.

*Subject to technical alterations.
Errors and omissions excepted.*

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Drive Medical Limited
Ainleys Industrial Estate, Elland
GB WEST YORKSHIRE HX5 9JP

Phone +44 1422-314488
Fax +44 1422-314489
Internet www.drivemedical.co.uk