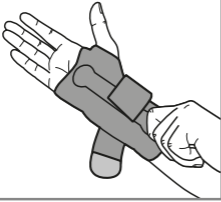
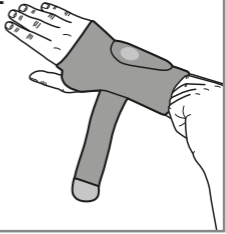
JuzoFlex
Manu Xtra
Handgelenkbandage
Wrist support

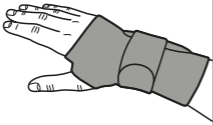
1

2

3

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Purpose

JuzoFlex Manu Xtra provides anatomical guidance for the affected wrist, thus relieving it during movement. Thanks to its bi-elastic knitted fabric in combination with anatomically shaped pads, the support applies compression, therefore promoting lymphatic drainage, which results in reduced swelling and pain alleviation. Proprioception and the resulting muscular auto-stabilisation of the wrist are improved. The adjustable plastic rod and the textile strap provide mechanical stabilisation.

Here is how to put on your JuzoFlex Manu Xtra correctly:

- 1.** Open the velcro strap and slide your hand into the wrist support until the stabilising splint lies on the inner surface of your hand. Put your thumb through the opening provided.
- 2.** Position the wrist support in such a way that the ring-shaped depression in the pressure pad encloses the wrist bone without pressure.
- 3.** Now close the velcro strap with sufficient tightness so that you feel an additional stabilisation of your wrist.

The correct fit of the wrist support is assured when it encloses the wrist tightly but does not apply a too strong (painful) pressure to the wrist.

The wrist support can be readjusted at any time by repeating steps 2 and 3 above.

Rough, unfiled nails or sharp-edged rings etc. may damage the very fine knitted fabric.

Please note:

Only wear your wrist support when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the wrist support can only be realised if it is applied correctly. The wrist support is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the wrist support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the wrist support is not suitable for treating more than one patient.

Washing and care instructions

Please observe the care instructions on the textile label attached to your wrist support. Please take the stabilising splint out of the wrist support and remove the strap. Close the velcro of the strap. Wash the wrist support and the strap in gentle or delicate cycle at approx. 30 °C. Wash the wrist support separately the first time (possibility of bleeding). Please do not use any softener! If washing by hand, rinse the wrist support well and allow it to dry at room temperature. If possible, lay flat. Do not wring it out. We recommend using the mild Juzo Special Detergent. To reduce the drying time, you can place the wrist support on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. This Juzo product must not be cleaned chemically. The removed stabilising splint can be cleaned with a damp cloth. Then slide it back into the pocket provided for it when the support is completely dry (with the bent spoon in front – the open side of the hollow section faces away from the hand surface). Finally velcro the strap onto the wrist support and it is ready for use again.

Material used

Details are provided on the label sewn into your wrist support.

Notes for the orthopaedic technician

The volar stabilising split is thermoplastically deformable at 180 °C. When shaping it, use suitable heat protection gloves to avoid injury.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

- distortions (sprains / strains)
- tendovaginitis (tendosynovitis)
- degeneration of the wrist (arthrosis)
- inflammation of the wrist (arthritis)
- post-traumatic irritation of the wrist

Contraindications

If the following symptoms exist, the support should only be worn after consultation with the physician:

Arterial insufficiency, lymph drainage disorders, post-thrombotic syndromes, skin disorders or skin irritation. Open wounds in the treated body area should be kept sterile.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritation) should occur while the use of our products has been prescribed, please go to your doctor or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your doctor before use. If your medical condition deteriorates while the use of our products, please go to your doctor immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the specialist dealer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your support can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nikakršne odgovornosti.
**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany