TOPRO TROJA 150 & ECCO

NO

ENG

DE

OTHER
LANGUAGES:
WWW.
TOPRO.NO

RULLATOR
ROLLATOR
ROLLATOR

BRUKERVEILEDNING
USER MANUAL
BEDIENUNGSANLEITUNG

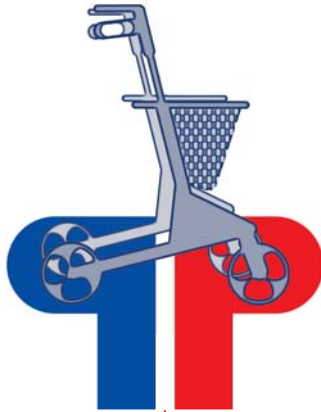
TOPRO
TOTENPRODUKTER AS
WWW.TOPRO.NO

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- 4-5 **Welcome**
 - Guarantee**
- 6-7 **Get to know your TOPRO ROLLATOR**
 - Measurement and weight**
- 8-9 **Important to know about safety**
- 10-11 **Assembling / adjusting**
 - Unfolding Folding**
 - Lowering the handlebars**
 - Height adjusting the handlebar tubes**
- 12-13 **Height adjustment using the memory function**
- 14-15 **Adjusting / checking the brakes**
 - Using the brakes**
 - Maintaining the brakes**
- 16-17 **Seat**
 - Basket**
 - Tilt function**
 - Edge guard**
- 18-19 **Material / recirculation**
 - Cleaning**
 - Disinfection**
- 20-21 **Storage**
 - Carrying the rollator**
 - Inspection / maintenance If the rollator doesn't work properly**
- 22-23 **Accessories**

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Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and made in Norway.
Manufacturer: TOTENPRODUKTER AS
Advanced manufacturing equipment, quality and environmental security gives great accuracy and maximum quality in all the levels of production.

Guarantee

TOPRO TROJA & ECCO
is guaranteed free for faults and defects for 5 years.

For repairs in the guarantee period, please refer to your local mobility shop, dealer or manufacturer.

An exception is if the damage is caused by incorrect use or parts that are exposed to natural wear and tear.

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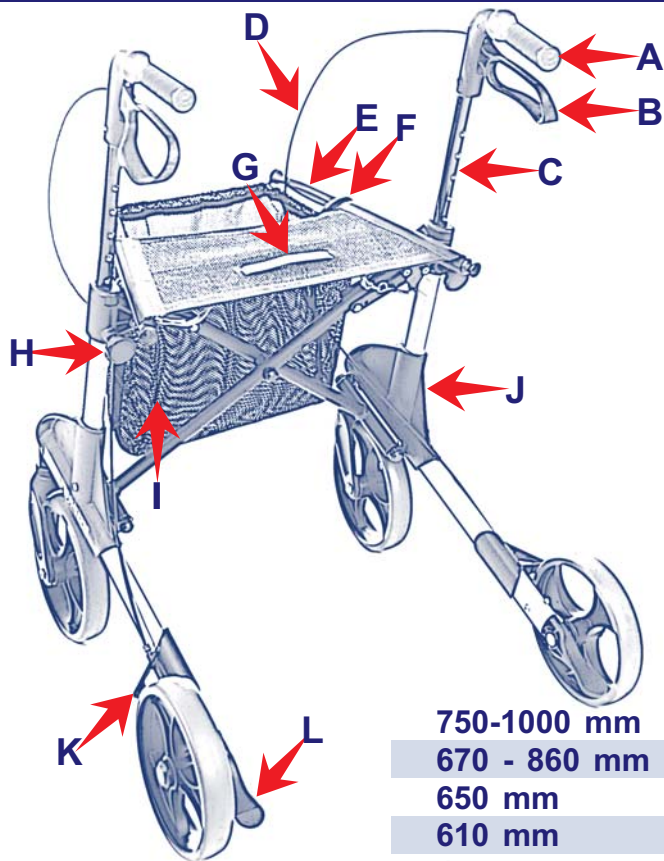
**Get to know your
TOPRO PRODUCT**

- A Handlebar**
- B Brake handle**
- C Handlebar tube**
- D Brake wire**
- E Seat bars**
- F Lock clamp**
- G Strap for folding**
- H Lock wheel for the handle**
- I Basket**
- J Side frame**
- K Brake block / Edge guard**
- L Tilt function**

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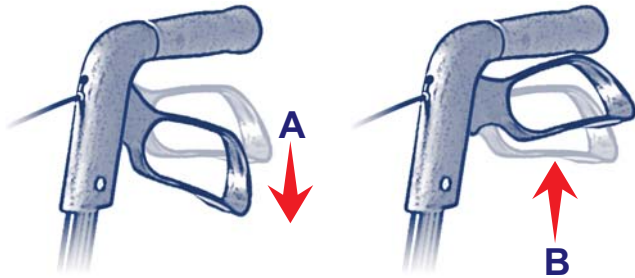
MEASUREMENT

- Height of support points TROJA**
- Height of support points ECCO**
- Maximum length**
- Maximum width**
- Between support points**
- Seat height TROJA**
- Seat height ECCO**
- Turning diameter**
- Diameter front/rear wheels**
- Width on front/rear wheels**
- Diameter handles**
- Grip distance driving brake**
- Height when folded up TROJA**
- Height when folded up ECCO**
- Width when folded up**
- Length when folded up**
- Weight TROJA**
- Weight ECCO**
- Maximum weight of user TROJA**
- Maximum weight of user ECCO**

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Important to know about safety

Before use:

- Notify your mobility shop, dealer or manufacturer immediately if the product is damaged when received.
- Not to be used by persons over 150 kg.
- The basket has a max load of 5 kg.
- Do not use to transport heavy loads or persons.
- Do not let children play with the rollator
- When folding and unfolding, be aware so you don't hurt your fingers.
- Check your brakes before use.

In use:

- When stopped or parked, always use the parking brake on *both* sides **A**
- Before sitting on the seat, make sure the parking brakes on *both* sides are on **A**
- The driving brakes can be held in a little **B** to reduce speed, but must not be in locked position when the rollator is in motion.
- To give the right support and a safe walk, make sure the rollator is kept as close to your body as possible **C**
- Walk straight, look forward and hold your backside in **D**
That is the easiest way to keep your balance.
- Do not push the rollator too far from your body as this can cause unnecessary strain and you could fall.

TOPRO TROJA and TOPRO ECCO are designed for in and outdoor use

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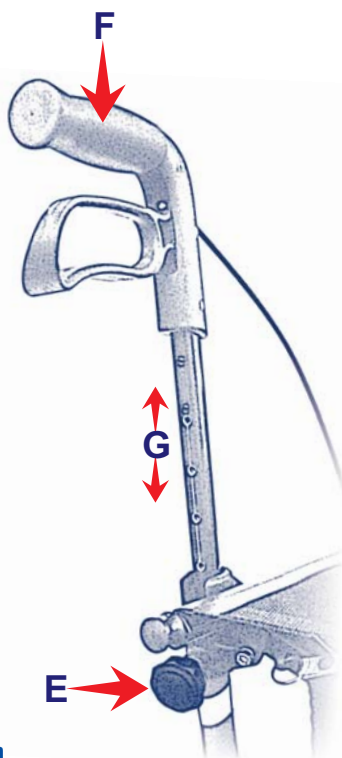
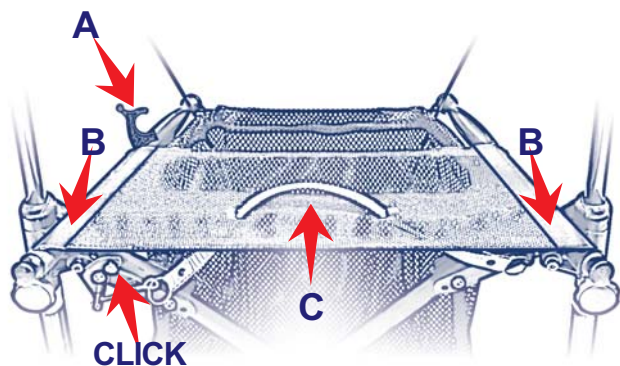
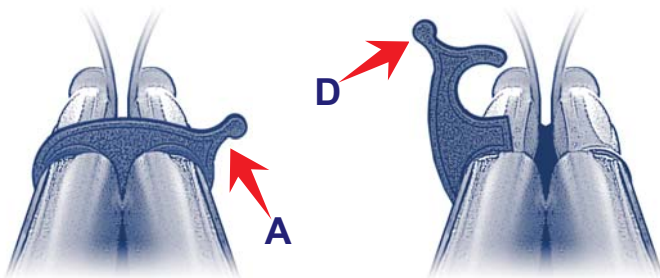
Assembling / adjusting

TOPRO TROJA & ECCO

are delivered assembled. Before use, you must unfold the rollator. The height of the handlebar tubes must be adjusted to suit the user. The handlebars should be the same height as the users wrists when the arms are straight down.

Unfolding

Loosen the lock clamp that is over the seat bars **A** Press the seat bars down and out **B** The side frames will slide apart and the seat bars are pressed down. You will hear a clicking sound. The rollator is now locked in an unfolded position.

Folding

Pull the strap on the seat **C** and press the side frames together. Press the locking clamp **D** over the seat bars.

Lowering the handles

Unscrew the lock wheel for the brake handles **E** (approx. 3-4 turns). Push the handlebar tubes down until they stop **F** Tighten the lock **E** again.

Height adjusting the handlebar tubes

Unscrew the lock for the brake handles **E** (approx. 3-4 turns). Adjust the handlebar tubes til a suitable height then pull them slightly up and down **G** until you hear a clicking sound. Tighten the lock **E** again. Adjust both the handlebars to the same height. The handlebars should be at the same height as the users wrist when the arms are straight down.

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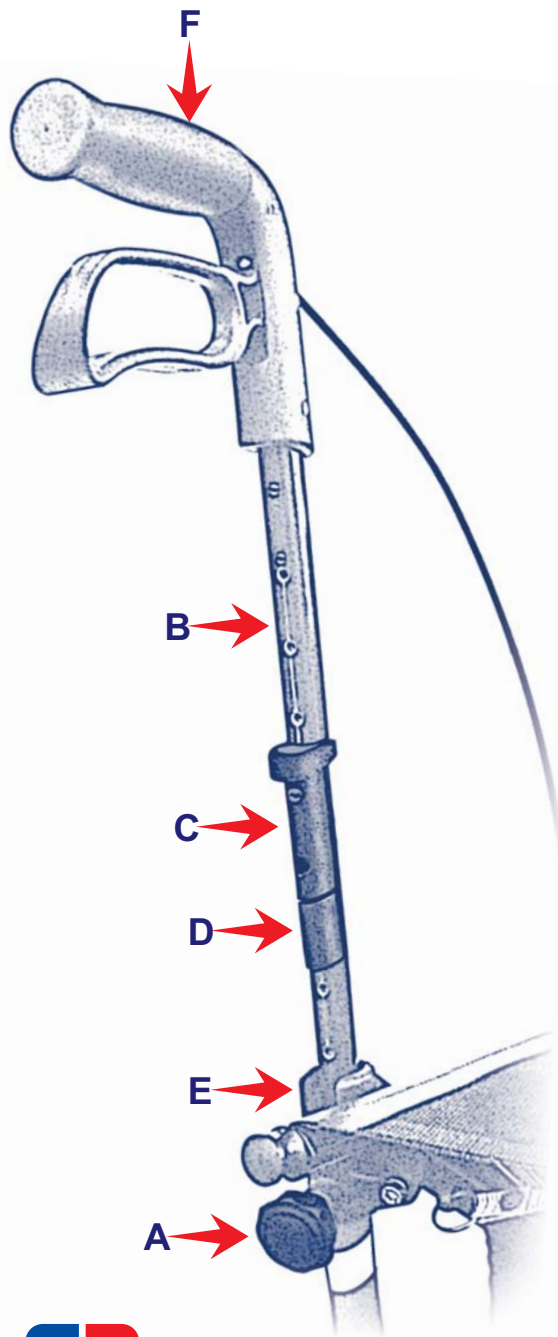
Height adjustment using the memory function

You have the opportunity to set the height of the handles as a standard so you easily can find the height you once set.

This can be necessary when the rollator is being transported or stored as compact as possible.

Decide the height of the handlebars. (They should be the same height as the users wrists when the arms are straight down.)

Count visible holes at the back of the handlebar tubes.

Unscrew the lock wheel **A**

Pull the tube **B** out of the frame.

The attached part **C** will now follow the handlebar tube. Move the attached part **C** on the tube **B** so there are equal holes as counted earlier. Fit the memory chip **D** on the handlebar tube in the first available hole under the attached part **C**

Push the handlebar tube **B** down and into the side frame **E** until the attached part is locked. Screw the lock wheel **A** half in.

Pull the handlebars **F** up until they stop, then tighten the lock wheel **A** properly.

From now on, when the handlebars are in the lowest position, they will stop in the preset position automatically when pulled up.

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Hjulene kan skiftes ved feil eller slitasje.

Adjusting / checking the brakes

Loosen the nut **A** beside the brake.
Tighten the brake wire **B** with the adjustment screw **C** Adjust both brake blocks so the distance to the wheel surface is approximately 2 mm **D**
Make sure the brakes are not adjusted too tight and pressing into the rubber when the parking brake is on.

Using the brakes

Make sure the brakes work before each trip. Each brake handle **E** works on each of the rear wheels. The rollator wheels are made from a special soft rubber which gives an optimal braking capacity on slippery surfaces.

Wrong use can cause unsafe situations and damage to the rollator: The rollator must not be driven or pushed with force when the parking brakes are on or when the wheels are partly locked using the parking brakes. This indicates incorrect use and can cause wear and tear which makes the tires and brake quality weaker.

Parking brakes: Push *both* brake handles **E** down until they lock. Both rear wheels are now locked. Squeeze *both* brake handles upwards to release the parking brakes.

Driving brakes: Pull *both* brake handles **F** upwards to reduce speed. Do not use continuously while the rollator is in motion.

Maintaining the brakes

Clean the brake block **G** and the brake spring **H** frequently. The wheels can be changed if they are worn out or damaged.

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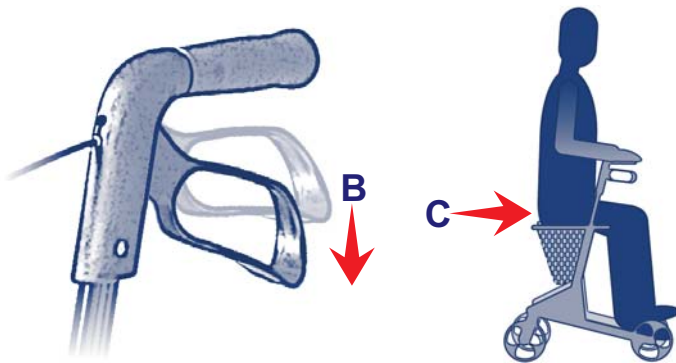
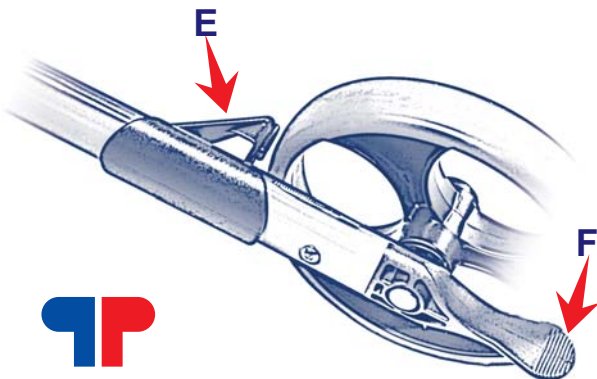
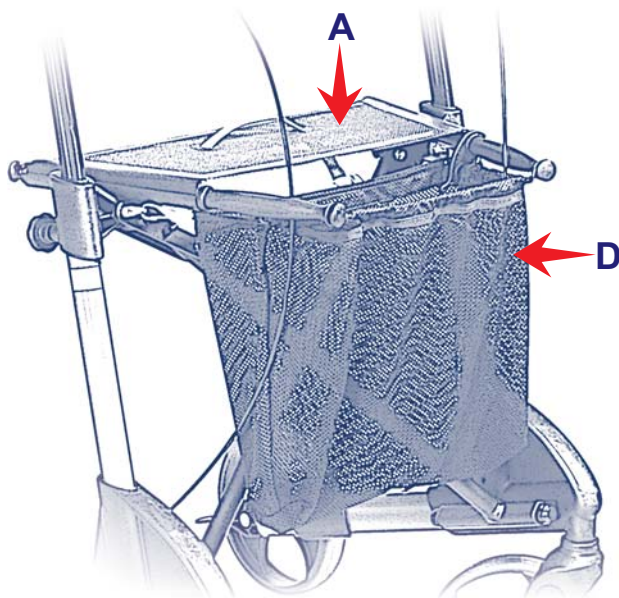
Seat

The parking brakes must be on before using the seat **A**
 Push *both* brake handles **B** downwards until they lock.
 Both rear wheels are now locked.
 Sit with your back towards the basket **C**
 The max user weight on the seat is 150 kg.

Basket

The basket **D** can carry goods up to 5 kg.
 Can be removed. Washes separately.

Edge guard

The brake block **E** which are in front of the rear wheels prevent the rear wheels from catching on corners, door frames etc.

Tilt function

Step on the tilt pedal **F** which is beside the rear wheel.
 Tilt the rollator over thresholds, edges etc.

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Material / recycling

The rollator is made of plastic coated aluminium pipes, plastic couplings and material of plastic and polyester. The most parts can be recycled.

Cleaning

The rollator can be steamed, hosed and washed with normal household detergents.

Disinfection must only be carried out by trained personnel

The surface must be cleaned with a disinfection that contains 70-80% ethanol.

We advise against disinfection that contains chlorine and phenol.

The manufacturer can not be held responsible for any damage or wounds that might occur through the use of harmful disinfection or disinfection carried out by unauthorized personnel.

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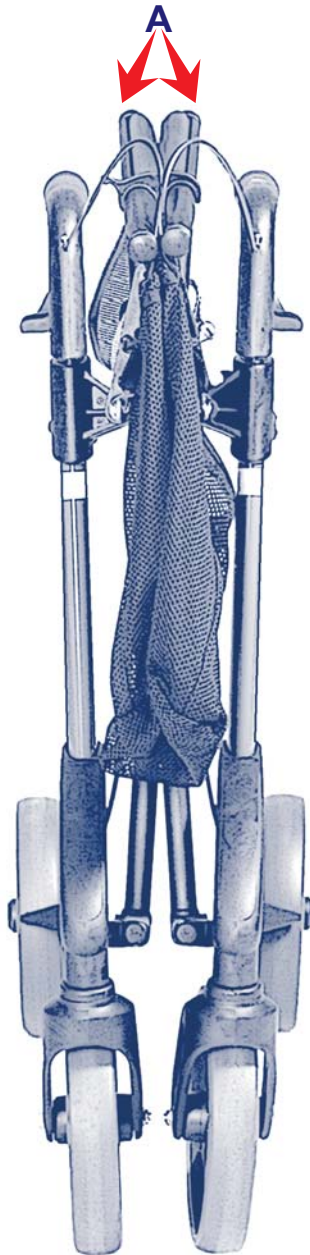
Storage

The rollator should be stored in an upright position. It can be stored outside.

Carrying the rollator

The rollator must not be carried by the seat strap or the brake wire. Fold the rollator together as shown on the illustration. Carry the rollator by the folded seat bars **A**

Inspection / Maintenance

It is recommended that general inspection and maintenance is carried out and that items below are checked on a regular basis:

- **Tube and frame**
- **Handlebar and handlebar tube**
- **Brakes**
- **Wheels**
- **Seat**
- **Accessories**

If the rollator does not work properly

- Make sure the rollator is unfolded correctly.
- Make sure that both brake handles are in expected position.

Incase of other faults or questions, please contact your nearest mobility shop, dealer or manufacturer.

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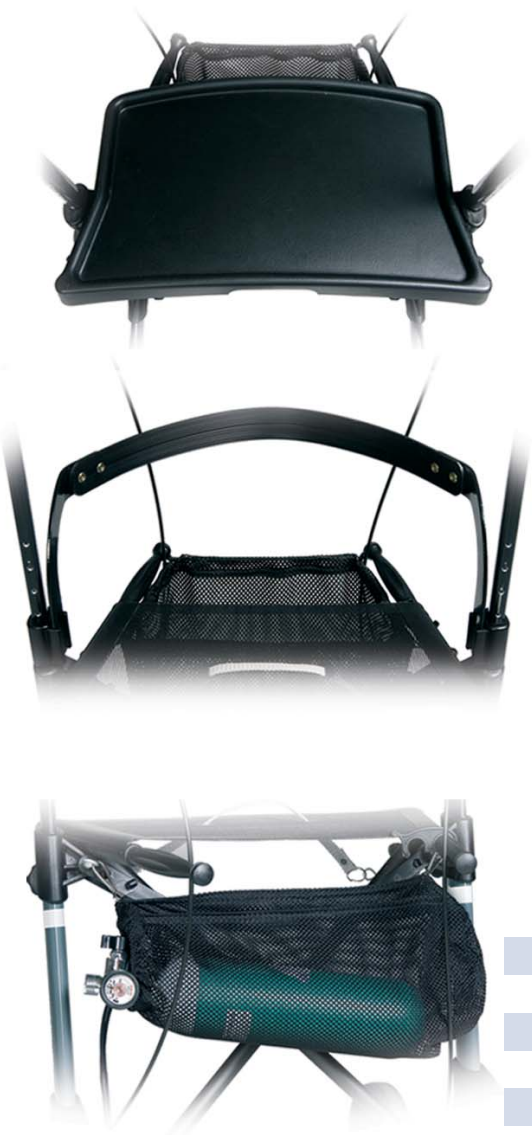
Accessories

To be ordered separately to give the possibility to compose a TOPRO TROJA and TOPRO ECCO that suits the individual users need. Here we have a range of products. Contact TOPRO for an updated overview of the accessories or visit our homepage.

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In case of faults or questions, please contact
your mobility shop, dealer or manufacturer.

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