JuzoFlex
Genu Xtra
Kniebandage
Knee support

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Purpose

JuzoFlex knee Supports provide anatomical guidance for the affected knee joint, thus relieving it during movement. Thanks to their bi-elastic knitted fabric, sometimes in combination with anatomically shaped pads, the supports apply compression, promoting lymphatic drainage which results in reduced swelling and pain alleviation. Proprioception and the resulting muscular auto-stabilisation of the knee joint are improved.

Here is how to put on your Juzo Knee Support correctly:

Please ask your physician or medical specialty retailer to show you how to apply your knee support correctly. We recommend that you follow these instructions closely. Attention: Rough finger nails, jagged rings etc. can damage the fabric.

Please note:

Wear your knee support only per your physician's prescription. Knee supports should only be dispensed by trained personnel at a medical specialty store. As therapy progresses or new symptoms appear, it will be necessary to take new measurements and possibly to produce a new knee support. Although your knee support is largely resistant to oils, ointments, skin moisture and other environmental influences, the fabric as well as a possibly worked in pressure pad can be adversely affected by certain combinations of these factors. It is therefore recommended to have your knee support checked regularly by your medical specialist retailer. If your knee support becomes damaged please consult your specialty retailer. Do not repair it yourself as this could impair the quality and effectiveness. For hygienic reasons, the support is not suitable for treating more than one patient.

Washing and care instructions

Please refer to the washing symbols on the label sewn into your knee support. Before washing, please ensure that any velcro fastenings are closed. Your knee support should be washed separately the first time (the colour might come out). Your knee support should be washed every day, if possible. Please do not use any softener! We recommend using the mild Juzo Special Detergent for this.

Material composition

Details are provided on the label sewn into your knee support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

- Arthritis in femoro-patellar joint
- Following surgery
- Knee joint distortions and effusions
- Following fractures
- Post-traumatic oedema
- Contusions
- To aid mobilization after prolonged immobility
- Sprains
- Subluxation and dislocation of the patella
- Meniscopathy

Contraindications

If the following symptoms exist, the support should only be worn after consultation with the physician: Arterial insufficiency, all forms of varicosity, following thrombosis, ulceration, recent or recurring flebothrombosis, permanent oedema of the lower leg. Skin diseases or skin irritations; Wounds in the treatment area are to be covered with sterile dressings. A below-knee compression stocking (AD) should also be worn by elderly patients with a tendency towards swelling and in conjunction with varicosity. The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your support can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany