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EN

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated. **The BORT RhizoFlex® thumb ring brace** is very pleasant to wear. Its integrated stabilisation element ensures optimum fitting

Instructions for use:

1. Slide the brace around your thumb with the Velcro tape on the back of the hand. Here, the aluminium stabilisation element surrounds the base of the thumb.
2. The Velcro tape Velcro fastener must be open, the tape however should remain in the opening.
3. Then close the Velcro tape.
4. As the aluminium stabilisation element is formable, this brace can repeatedly be refitted during the course of therapy in order to retain an optimum fit.

The brace can be used in water without any problems. If it comes into contact with saline water, rinse it afterwards with ample fresh water.

Indications:

Post-operative, post-traumatic or degenerative irritations in the thumb saddle joint region, e.g. arthrosis of the CMC-I joint (thumb saddle joint), instability of the CMC-I joint, post-operatively following arthroplasty of the CMC-I joint, instability of the CMC-I joint.

Contraindications:

Lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition: Plastic frame: 70 % thermoplastic polyurethane, 30 % thermoplastic elastomers
Sealing tape: 55 % polyester, 45 % polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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