ART.-NR. 07074

MALLEO-HIT®

ART.-NR. 87074

MALLEO-HIT® KIDS

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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

MALLEO-HiT® helps to relieve, compress and stabilise the ankle joint. The integrated knotted silicone pressure pads improve the circulation in the joint area by means of precision friction **1**.

This enhances the supply situation for the cartilage and speeds up the absorption of effusions. The friction also results in better muscular stabilisation.

INDICATIONS

- Femoral/Crural arthrosis.
- Rheumatism.
- Everyday preventive treatment for weak ligaments.

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

- MALLEO-HiT® can be worn on the left or right ankle.
- Take the upper part of the bandage and wrap it around up to the top edge of the silicone profile padding.

• Pull the bandage over the foot until the heel fits properly in the anatomically stitched heel section. The silicone profile padding should then be positioned behind and beneath the medial/lateral malleolus.

• Wrap the upper part of the bandage around again and, if necessary, pull the bandage upwards a little for a proper fit.

• MALLEO-HiT® is fitted properly when the seam runs over the middle of the foot and heel.

COMFORTABLE TO WEAR

MALLEO-HiT®'s special stitching in the instep area makes for an almost crease-free fit during heel-toe walking, substantially improving comfort when wearing.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

Wash MALLEO-HiT® in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. The enclosed heel pad can be cleaned with a moist cloth.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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Vsi izdelki družbe SPORLASTIC GmbH se preverjajo v okviru našega sistema za upravljanje kakovosti. Če bi naš izde-