ART.-NR 07028

RIPPENGÜRTEL

SPORLASTIC

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Rib support for men

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Faja costal para caballeros

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INSTRUCTIONS FOR USE GB

Dear Customer, we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

The encircling pressure exerted by the rib support limits the breathing-induced expanding movement of the rib cage, thereby helping to diminish pain.

INDICATIONS

- _ Rib bruising
- _ Rib fractures
- _ Breast bone bruising
- _ Breast bone fractures

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- _ Simultaneous bronchitis and pneumonia.
- _ Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- _ Impaired circulation or lymphatic soft tissue swelling.
- _ Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped body fitting.

FITTING

- _ Place the rib support around the chest with the expanding insert at the back and the Velcro fastener at the front.
- _ Pull the two ends tightly together and apply the fastener in such a manner that shallow breathing is still possible.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in conjunction with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

Wash rib support in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. The enclosed heel pad can be cleaned with a moist cloth.

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