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67% Polyamid, 33% Elasthan

INSTRUCTIONS FOR USE GB

Dear Customer, we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

ACHILLO-HIT®'s special knopped silicone truss pad achieves a compression and massaging effect on the Achilles tendon. The silicone pad's insole pressure cushion exerts proprioceptive stimuli on the tendon of the posterior tibial muscle and enhances the foot's pronation control. The conscious decision to abandon a heel raiser means that the calf muscles are not contracted.

INDICATIONS

Conservative treatment:

- Achillodynia (pain in the Achilles tendon)
- Tendinitis / Peritendinosis / Peritendinitis of the tendon
- Paratenonitis
- Achilles bursitis
- Sub-Achilles bursitis

Postoperative treatment:

- Operations on the gliding tissue of the Achilles tendon
- Sub-Achilles bursitis operations
- Partial rupture of the Achilles tendon
- Achilles tendon rupture
- Reconstruction of the peroneal tendon sheath or the peroneal tendon
- Reconstruction of the posterior tibial tendon sheath or the tendon

CONTRA-INDICATIONS

- Severe arterial circulation disorders
- The bandage does not replace the function of the Achilles tendon

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

– Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.

- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped body fitting.

FITTING

There is ACHILLO-HIT® for the left foot and the right foot.

- Open the Velcro fasteners to slip into ACHILLO-HIT®
- Grasp the sides of the Velcro fasteners with both hands. Pull the bandage over the back of the foot, gently at first. Then hold the back of the bandage and pull it over the heel.
- The seam of the bandage has to be flush with the middle of the back of the foot/the heel bone.
- **IMPORTANT:** The integrated knopped silicone truss pad has to be placed in such a way that it can have a massaging effect on the side near the Achilles tendon. No knobs should massage the Achilles tendon.
- The truss pad's longer pressure cushion on the inside of the foot has to extend beneath the medial malleolus right into the arch of the foot (Cover picture). The shorter outside pressure cushion also extends beneath the lateral malleolus.
- Close the Velcro fasteners in the region of the front lower legs.

COMFORTABLE TO WEAR

The special stretchable stitching in the instep area developed by SPORLASTIC makes for an almost crease-free fit during heel-toe walking, clearly improving comfort when wearing.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

Wash ACHILLO-HIT® in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. Close Velcro fasteners before washing as this will lengthen their useful life and avoid damage to other laundry.

TIP: Slackening tension of the knitted fabric while it is being worn is reversed by washing. The bandage should therefore be washed frequently.

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