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Back Supports support injured tissue structures (bones, ligaments and cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the vertebrae. Fit and height are adapted to the female anatomy. The pad provides a massage effect which reduces swelling tendencies or which helps to an earlier restraint of swelling.

COOLMAX® regulates the body temperature to provide a dry and pleasant feeling on the skin.

### **Instructions for use:**

Place the support around the waist and – using the hand straps – evenly overlap the two Velcro fastening ends by tensioning slightly. Ensure that the support is centered and that the lower edge of the support is in the groin area (forward bend). This is important for optimal effect of the BORT StabuloBasic. The pad is velcroed to the support and can be positioned freely. Ensure that the wide side of the pad faces upward.

**Indications:** Treatment of increased pain in the lumbar spine area, severe low back pain, recurrent low back pain, osteochondrosis, spondylarthrosis, lumbosciatica, mild radicular syndrome, spondylolysis without spondylolisthesis, mild to moderate lumbar deformity (degenerative spondylolisthesis), after intervertebral disc surgery.

**Contraindications:** Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

### **Composition:**

46 % Spandex, 30 % polyamide, 24 % polyester

### **Important instructions:**

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

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The **BORT StabiloBasic Lady Sport** unites the stabilizing function of a back support with the moisture regulating properties of the COOLMAX® high-tech fiber. Innovative materials and the most modern of production technology give the **BORT StabiloBasic Lady Sport** optimal properties for sports-related and therapeutic applications.

**BORT-Quality products** promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | [www.bort.com](http://www.bort.com)