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INSTRUCTIONS FOR USE GB

Dear Customer,

We would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

RHIZO-HIT® braces the thumb in a functional abduction and opposition position.

An individually anatomically pre-formed aluminium insert stabilises the base joint and the saddle joint of the thumb. The terminal phalanx of the thumb stays free to move, enabling the key grasping function.

INDICATIONS

- Lesions, ligament instabilities in the region of the thumbfundus and thumb saddle joint (e.g. 'ski thumb')
- Arthrosis
 - of the radial carpus
 - of metacarpophalangeal joint 1
- Root arthrosis

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic

disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

FITTING

RHIZO-HIT® can be worn on the left and the right thumb.

1

- Close the Velcro fasteners at the end of the thumb tube, so that the opening fits tightly but nevertheless comfortably around your thumb.

2

- In order to achieve an optimal fit around the thumb, the hook side of the Velcro should be placed congruently over the fleece strip without protruding over the edges.
- Slip the thumb requiring support into the bandage. The side enlargements on the strap prevent it slipping out of the guide loop, which at the same time also allows easier access.

3

- Take hold of the non-elastic strap with the thumb and index finger on the Velcro surface and fix RHIZO-HIT® by closing the strap at the wrist joint.
- Using a minimum of effort, the guide loop allows the tension in the strap to be varied in order to stabilise the bandage at the wrist.
- Adjust the tension of the strap, so that there is noticeable stabilisation of the thumb and wrist and it continues to be pleasantly comfortable to wear. A possible readjustment of the Velcro around the thumb may be necessary.

4

- The aluminium reinforcement should be individually adjusted at the first fitting in the specialist dealership. You can undertake further corrections to the fit yourself if this is necessary, by respectively pressing the bandage and changing the shape of the aluminium insert in the worn state using the other (free) hand.

5

- Fig. 4 shows how RHIZO-HIT® should fit properly.

Note for the orthopaedic technician: After the initial fitting, RHIZO-HIT® has to be adjusted to provide an optimal fit by pressing the integrated aluminium reinforcement to the natural shape of the hand. By changing the shape of the aluminium insert, individual and different corrections to the position of the thumb can be achieved, according to the respective indication. The change of shape can be simply carried out manually without the need for any special tools.

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