PATELLADYN®
SPORLASTIC

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INSTRUCTIONS FOR USE GB

Dear Customer,

We would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

Active muscular and passive patellar medialisation.

Knee bandages are very often assessed according to their passive, i.e. external supporting effects such as via the stitching, pads and joint splints. The findings from Neurophysiological Research in recent years substantiate the active effective components of bandages.

With the PATELLADYN® Knee Bandage, the lower strap hanging downwards exerts pressure on the patellar tendon ligament by means of the silicone ring section located beneath the kneecap. This brings about quicker and more powerful stimulation of the thigh muscles. This means that the kneecap can be guided/kept in the right position by active use of the muscles. The kneecap is also passively centred via the silicone halfring by the inward pulling force 1.

INDICATIONS

- Patella lateralisation (subluxation) with
 - _ Chondropathy patellae
 - _ Gonarthrosis
- Recurring and habitual patella luxation
- Cruciate ligament insufficiency

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- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general

side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped body fitting.

FITTING

- Draw the PATELLADYN® knee bandage over the knee in such a manner that the two silicon profile inserts (large and small silicon ring) encircle the knee-cap.

- The larger semi-circle part of the silicon ring must encircle the knee-cap from the outer side of the knee.

- The two tightening cords of the knee bandage are connected to this profile insert 2.

- Conduct the two cords with a gentle pull around the knee 1.

- Secure the cords with the hook-and-loop fasteners.

COMFORTABLE TO WEAR

Comfortable to wear due to the almost ideal crease-free fit as a result of the concertina action in the hollow of the knee: The core of this invention is the oval intarsia insert involving an anatomical flat-knit fabric without any disturbing seam in the hollow of the knee. The patient does not feel the side seams.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability. Please make sure to always close the Velcro fasteners properly. As a matter of principle, damage arising from improper fastening of the Velcro fasteners is not accepted as grounds for claims.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

Wash PATELLADYN® in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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