SchnarchStopp – Die Anti-Schnarch-Bandage

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SchnarchStopp – The Anti-Snore-Brace

Breathing through the mouth in a supine position makes the respiratory tracts constrict and frequently causes snoring. The Anti-Snore-Brace SchnarchStopp promotes lying in a lateral position and thus enables to sleep without snoring. The brace consists of a pressure device made of foam (Wendestopp) with two plastic plugs and a bandage.

Indication

- Sleep disorders caused by snoring

No contraindications are known.

How to apply

- Unfold the brace and insert the Wendestopp with the aid of the enclosed pouch into the pocket. Then close the pocket with the hook-and-loop fastener to secure the position of the Wendestopp.
- If necessary you can enhance the effect by putting the two red plastic plugs into the holes of the Wendestopp. Make sure that they do not press on the spine.
- Put the brace around the chest and close it on the front side with the hook-and-loop fastener. It can be worn both over and under the nightwear.

Important informations

- Do not use the brace any longer if you keep on lying in a supine position. Consult a doctor if there are no signs of improvement though using the brace since the reasons for the sleep disorder might be profounder.
- Do not use the plastic plugs when suffering from osteoporosis or back problems!
- For hygienic reasons used braces cannot be returned!

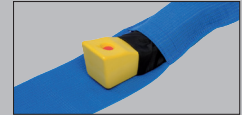
Materials

- Brace: polyamides, other fibres, polyesters, cotton, polyurethane
- WendeStopp: PUR foam, plastic, polyester

Washing instructions

- Close the hook-and-loop fasteners before washing
- See care label

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