
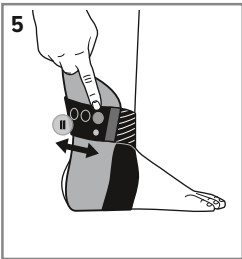
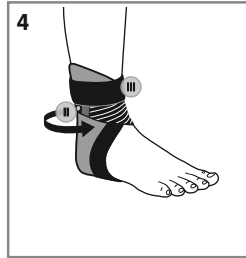
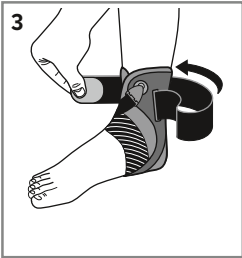
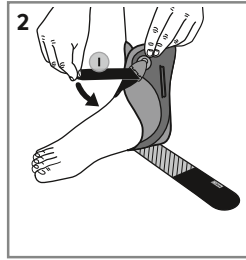
JuzoPro

Malleo Xtec Strong

Sprunggelenkorthese · Ankle orthosis

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкција по експлуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

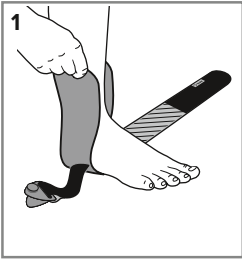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EN

Intended purpose

The JuzoPro Malleo Xtec Strong secures the upper and lower ankle in the frontal plane using a medial stabilisation element in combination with an adjustable strap system. It limits the pronation and supination movement, i.e. inward and outward twisting, as well as talar shift.

This is how to put on your orthosis

JuzoPro Malleo Xtec Strong correctly:

Put on your orthosis while sitting down to prevent injuries or falling.

1. Open the touch fastener on the strap with the green Juzo logo and fully pull the strap out of the diverter. Open the touch fastener on the other strap and remove the clip from the fastener in the stabilisation element. Now position your foot in the orthosis in such a way that your heel is in the specified opening and the seam runs parallel to the Achilles tendon. Your foot should be at a 90° angle to your lower leg (Fig. 1).
2. Now diagonally place the narrow strap with the clip across your forefoot and use the clip to secure it in the fastener. Close touch fastener I to exert firm, yet pleasant tension (Fig. 2).
3. Then loop the wide, colourful strap through the diverter provided (Fig. 3). Tighten this strap in the same way to create firm, yet pleasant tension.
4. Finally, wrap this strap back around the touch fastener section next to marking II and attach it using touch fastener III (Fig. 4). The correct fit of your orthosis is guaranteed when it firmly surrounds the ankle without applying too much (painful) pressure. You can re-adjust the orthosis at any time by repeating Steps 3 and 4.
5. Optional step: the touch fastener section next to marking II can be removed and positioned along the upper edge of the strap as required to adjust the height and tension of the strap (Fig. 5).

This product can be used barefoot as well as over socks. In order to guarantee maximum stabilisation, the orthosis should be worn in closed shoes.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The orthosis can only be fully effective when it has been put on correctly and is being worn in accordance with treatment recommendations. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the product. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient. The welded-on label on the foot base is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

Washing and care instructions

Please note the care instructions on the inside of the product. Wash the orthosis separately the first time (colours may run). Wash the orthosis with laundry detergent for delicate fabrics every day or between uses at approx. 30 °C. We recommend using the mild Juzo Special Detergent for this. Then rinse it well and leave it to dry at room temperature. You can reduce the drying time by placing the orthosis on a thick towel and dabbing it dry. Do not leave it in the towel and do not dry it on a radiator or in the sun. The orthosis is not suitable for tumble drying. This Juzo product is not suitable for dry cleaning.

Material composition

For full details, please refer to the label on the inside of the product.

Storage information and usage duration

Store in a dry place and protect from sunlight. The medical retailer or doctor prescribing the orthosis will advise on exactly how long it can be used for.

Indications

Chronic ankle joint instability, mild soft tissue injuries, capsular/ligament injuries, following ligament lesions and/or sprains, stabilisation of the ankle during exercise, post-operative stabilisation.

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician: lymph drainage disorders, arterial flow impairment, post-traumatic conditions, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings. The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly.

Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com
CH REP Berro AG · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd