ART.-NR. 07057
MANUDYN®

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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

MANUDYN® stabilises the wrist in an axially and functionally correct neutral zero position.

The partial stabilisation of the wrist is made possible by the knitted fabric, an anatomically shaped reinforcement plate on the inner side of the hand and a circular, finely adjustable Velcro strip around the wrist.

The anatomical silicon profile pad around the styloid process of the wrist supports the circulation, improves the resorption of oedema and haematoma and prevent pressure irritation.

The use of latex-free materials provides continuous compression and makes the bandage very comfortable to wear.

INDICATIONS

- Acute phase of active arthritis in the wrist.
- Following moderate to severe distorsions.
- Tendon sheath inflammations.
- Wrist pains caused by instability and hypermobility (arthralgia).
- Stress related arthropathy.
- Interim care after the removal of a cast (follow-up care of wrist fractures).

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.

Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped body fitting.

FITTING

Open the circular Velcro strip before putting on the MANUDYN® wrist bandage.

Slip your hand into the MANUDYN® and place the thumbs through the thumb openings provided.

MANUDYN® will fit properly if the reinforcement plate is positioned tightly against the musculature of the ball of the hand and the underarm **1**. The recess for the silicon pad will then surround the bony styloid process of the ulna located on the little finger **2**.

Pull the wide elastic Velcro strip circularly around the wrist for additional stabilisation **3**.

Open the circular Velcro strip to remove MANUDYN®. Reach into the bandage from the inner side of the hand with the index and middle fingers. Pull the bandage from the hand by pressing the thumb against the reinforcement plate.

WEARING COMFORT

MANUDYN® is manufactured from high tech materials and combines modern design with function and wearing comfort.

The anatomically knit shape with thumb opening prevents slipping and provides an optimal position for the bandage.

An especially flexible knit fabric optimally adapts to the contours of the wrist, thus making it very comfortable to wear.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

Please make sure to always close the Velcro fasteners properly. As a matter of principle, damage arising from improper fastening of the Velcro fasteners is not accepted as grounds for claims.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system.

If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

We recommend that MANUDYN® be washed gently in warm water with a mild detergent and air dried, not place on a heater.

Please remove the stabilisation plate from the plastic before washing. Close the circular Velcro strip before washing. This will allow the Velcro to remain functional longer.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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Wszystkie produkty firmy SPORLASTIC GmbH podlegają kontroli w ramach naszego systemu zarządzania jakością.
SPORLASTIC GmbH Medizinische Produkte