NEURODYN®-CLASSIC SPORLASTIC

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INSTRUCTIONS FOR USE GB

Dear Customer,

We would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

The NEURODYN®-CLASSIC foot levator orthotic device enables free plantar flexion of the foot for a good return of the foot during the swinging phase while walking, thereby ensuring an almost completely natural movement.

The two semi-elastic bridles (1, 2) support foot lifting. The 8-shape bridle lacing stabilizes the ankle joint. The non-elastic side bridle (3) provides lateral support and helps to prevent upper ankle sprains.

Applying bridle 3 (5)
Finally conduct the non-elastic bridle 3 with an upward pull around the outer side of the ankle. Lift the outer edge of the foot slightly with the bridle. Then secure bridle 3 to the large fastener of bridle 2.

Applying bridle 2 (4)
The elastic, dynamic design ensures proprioception since the sole of the foot is hardly influenced while walking, thereby significantly promoting balance and co-ordination.

The materials of the product are latex-free!

INDICATIONS

- Paralysis of the peroneal nerve
all degrees of insufficiency, especially in cases of paraparesis of the levator of the foot and the toes, light to moderate (degree of strength 2 - 4)

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- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, redness) of the body areas being treated.

- Impaired circulation or lymphatic soft tissue swelling.

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Excessive lengths of bridle 1 and 2 can be cut off.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability. Please make sure to always close the Velcro fasteners properly. As a matter of principle, damage arising from improper fastening of the Velcro fasteners is not accepted as grounds for claims.

FITTING

Applying the NEURODYN®-CLASSIC foot levator orthotic device:

Ideally, the NEURODYN®-CLASSIC foot levator orthotic device is worn directly on the skin, but it can also be worn over a tight-fitting stocking.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

TIP: To simplify the application ensure that the foot is dry

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

Wash NEURODYN®-CLASSIC in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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