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INSTRUCTIONS FOR USE GB

FUNCTION

VERTEBRADYN®-STABIL is an orthosis for the stabilisation of the lumbar column. It incorporates at the back anatomically pre-shaped spring steel rods.

The 3-dimensional anatomic, elastic and freely breathing knitted bandage ensures functional compression at maximum wearing comfort.

The combination of abdominal support with rear stabilisation elements relieves the vertebral column and straightens the pelvis.

The compression pressure of the VERTEBRADYN®-STABIL can be individually adjusted with the continuously variable touch fastener. The loops around the fastener ensure that the VERTEBRADYN®-STABIL can be very comfortably fitted.

INDICATIONS

- _ Lumbar column syndrome
- _ Osteochondrosis
- _ Spondylolisthesis
- _ Instability of the vertebral column
- _ Stable osteoporotic fractures
- _ Lumbago
- _ Spondylosis
- _ Intervertebral disk damage
- _ Radiculopathy and ischialgia
- _ Preventative stabilisation, e.g. during sports and physical work

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- _ Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- _ Impaired circulation or lymphatic soft tissue swelling.
- _ Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

- 1 _ The correct position of the VERTEBRADYN®-STABIL has been achieved when the stabilising element in the back section is centred over the lumbar column.
- 2 _ When seated, the front lower end of the bandage should be level and parallel with the thighs and not cause any pressure on the groin.
TIP: Wear the VERTEBRADYN®-STABIL over a tight fitting T-shirt or vest.
- 3 _ To fit pass your fingers through the loops at both ends of the faster of the VERTEBRADYN®-STABIL. Pull the orthosis apart behind your back and slip it over your body.
- 4 _ Tighten and close the fasteners across your abdomen. Pull your fingers out of the loops and close the fastener of the VERTEBRADYN®-STABIL.
- 5 _ Ensure when closing the fastener that the fastener tabs are perfectly aligned over each other.
TIP: VERTEBRADYN®-STABIL should exert a high compression on the body. The combination of the stabilising elements in the lumbar vertebra area with abdominal support should achieve a distinct relief of the vertebral column. Ensure that the VERTEBRADYN®-STABIL is comfortable when it is fitted at a high compression rate.

NOTE FOR THE ORTHOPAEDIC TECHNICIAN

The anatomically pre-shaped spring-steel rods can be individually adapted to the patient.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

Wash the orthosis in hand-warm water with a mild detergent and then dry in the open air. Do not place it on a radiator! To prevent damage to clothes wash the orthosis separately or close the touch fasteners beforehand.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The orthosis should therefore be washed frequently.

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Vsi izdelki družbe SPORLASTIC GmbH se preverjajo v okviru našega sistema za upravljanje kakovosti.