LUMBO-HIT®
SPORLASTIC

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QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

We recommend that LUMBO-HIT® be washed gently in lukewarm water with detergent for delicate items and that it then be air dried (do not dry on a heater).

To prevent damage to other items of clothing when washing, it is better to wash LUMBO-HIT® separately or close the Velcro fasteners beforehand.

INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

LUMBO-HIT® is an active lumbar bandage made of lightweight, breathable material. LUMBO-HIT® should be fitted with deliberately high compression and is equipped with two extra elasticated drawstrings to support and stabilise the lumbar spine **1**.

Used with a semi-flexible, knopped pad in the region of the lumbar spine, LUMBO-HIT® can actively take the strain off and prevent backward curvature of the lumbar spine **3**.

This can produce positive effects like reducing pain segmentally, centralising the body's centre of gravity to decompress the intervertebral discs and stabilise the back muscles.

INDICATIONS

- Lumbar spine syndrome
- Lumbago
- Muscle imbalances
- Irritation of the sacroiliac joint
- Pseudoradicular radiation of pain
- Irritation of the minor facet articulations (facet syndrome)

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).
- So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been

no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING (right seat see picture)

– The LUMBO-HIT® bandage with the pad fastened inside is correctly positioned when you hold the two ends behind your back in such a way that the back pad is centralised over the lumbar spine. Fasten the bandage over your navel using the Velcro fastener.

– The lower end of the LUMBO-HIT® should be level with the thigh when you are seated and not cause any pressure to bear in the flexure of the groin.

– The bandage can be repositioned as and when required by opening the Velcro fastener. When closing the fastener, make sure that the two ends of the bandage are fastened parallel to each other.

– Now take the additional left drawstring, pull it forcefully forward and downwards in a diagonal direction and fasten it onto the adhesive area of the bandage.

– Pull the right drawstring forcefully upwards in a similar fashion and fasten it onto the left drawstring **2**.

– To achieve the optimum effect (taking the strain off and stabilising the lumbar spine), you should deliberately fit LUMBO-HIT® with high compression.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

We recommend that LUMBO-HIT® be washed gently in lukewarm water with detergent for delicate items and that it then be air dried (do not dry on a heater).

To prevent damage to other items of clothing when washing, it is better to wash LUMBO-HIT® separately or close the Velcro fasteners beforehand.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

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