COXA-HIT

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GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

Function

COXA-HIT is a hip bandage used for conservative or postoperative treatment of diseases of the hip joint.

The breathable elastic material stabilizes the hip joint and improves the coordination of the hip muscles.

The enclosed cold/hot pad is used, following a physician's instructions, for thermotherapy and to aid in compression. Pain and swelling may be reduced.

Tip: For postoperative use, COXA-HIT can be combined with an anti-thrombosis stocking.

Indications

■ **Postoperative**, e.g., after hip operations (endoprotheses, implantation or replacement).

■ **Conservative**, e.g., for arthritis, muscle strains.

Contra-indications:

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

■ Long-lasting, fixed immobilization, particularly in elderly persons.

■ Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.

■ Impaired circulation or lymphatic soft tissue swelling.

■ Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

Side-effects:

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

Fitting

■ Open the Velcro fasteners of the bandage.

■ Loosely fasten the upper hip belt to the side to the large fleece surface on the leg portion.

■ The hip belt of the bandage is applied approximately 5 cm below the belly button.

■ To do this, use your fingers to grasp the grip provided at the end

• The bandage is applied in such a way that it provides comfortable compression. Now take the upper hip belt that was previ-

ously loosely fastened at the side and pull it forcefully through the belt below it. When you do this, the silver compression control strip should be fully tensed (see **1** and **2**).

■ To apply the leg portion, use the grip on the inner upper leg belt.

• The upper leg belt is now closed in a circular fashion.

• The silver compression control strip that is attached here will be fully tensed if applied correctly.

■ The enclosed cold/hot pad is, in accordance with the doctor's instructions, used for thermotherapy and to increase the compression effect.

Note: Please see the hints for use for the enclosed cold/hot pad.

SKK (Sporlastic Compression Control Strip)

1: COXA – HIT applied under tension.

2: COXA – HIT applied too loosely.

Using the Sporlastic Compression Control Strip, the physician, patient, or nursing staff can easily determine whether the hip bandage was applied in accordance with its function.

Important:

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

SPORLASTIC GmbH is affiliated to Duales System Deutschland GmbH return and recycling system, "The Green Spot", and thus meets the requirements of the Packaging Order applicable in the Federal Republic of Germany.

Please dispose of your used packaging at the local recycling depot.

Quality Management System

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

Care

We recommend that COXA-HIT be washed gently in lukewarm water with detergent for delicate items and that it then be air dried (do not dry on a heater). Close the Velcro fasteners before washing. If you do so, the fasteners will stay functional longer and damage to other clothing items can be avoided.

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