
ART.-NR 07031 - 07032

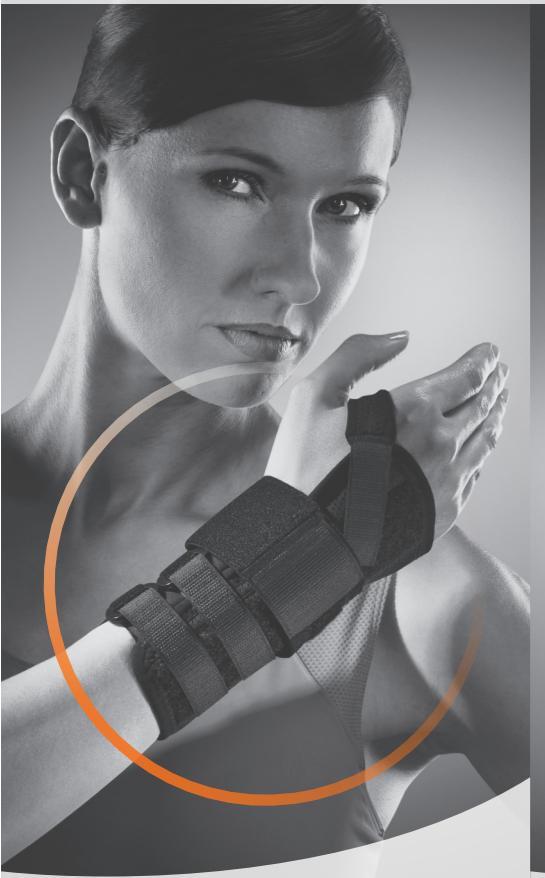MANU-HiT® CLASSIC SPORLASTIC

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So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

FITTING

– Slide your hand into the brace 1.

– Guide the two narrow Velcro tapes 2 in each through the eyes and firstly fix the brace to your hand with slight tension by pressing the tapes together. The brace has the correct fit when the volar splints fits snugly against your hand an lower arm.

– Now guide the Velcro tape 3 between your thumb and forefinger and fix this.

– The inelastic wide Velcro tape 4 reinforces the stabilization of the wrist by dorsal circular tension. This step is not necessary with MANU-HiT® S.

– Finally put on the Velcro tapes 2 so that the brace sits firmly against your hand without pressing unpleasantly. If necessary correct the Velcro tape 3 as well.

– The straps can be shortened if necessary.

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IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

FUNCTION

MANU-HiT® CLASSIC and MANU-HiT® S have a stabilising, movement restricting function for the wrist joint. This is achieved by volar (on the inside of the hand) and dorsal (on the back of the hand) stabilizing splints. In this way the loads on the wrist bones, on the articular capsules and on the ligaments is reduced.

The wide volar splint is anatomically preformed to stabilize the hand and can be adapted individually as required. The narrow dorsal splint is introduced into the pockets in the back of the hand for additional stabilisation either more medially or laterally – towards the inside or outside of the hand. In this case the splint should be inserted in the fold at the open end of the velour band to prevent the splint slipping out in use.

MANU-HiT® S has an additional integrated, anatomically shaped silicone butterfly pad that promotes blood flow and the resorption of edemas through a targeted massage effect.

CARE

Wash MANU-HiT® CLASSIC and MANU-HiT® S in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. The enclosed heel pad can be cleaned with a moist cloth. When washing please remove the two splints from the brace. Close the Velcro fasteners before washing. The Velcro fasteners will then last longer.

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CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated
- Impaired circulation or lymphatic soft tissue swelling

- Neurogenically-specific organoleptic and skin trophic disorders with and without skin damage

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