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The first choice for most demanding people.

select[®] - our highest-quality series of supports convinces for its functionality and the excellent product components. It offers unique characteristics, which satisfy even the most demanding patients.

BORT. Benefit on your side.

GB Dear Customer,

Thank you for choosing a premium medical product manufactured by BORT GmbH. With the **BORT select® GenuZip® Knee Support**, your specialist medical provider has supplied you with a product that meets the highest medical, functional and quality standards.

All of the products in the BORT **select®** Support series feature **TIMBRELLE® CARE** antimicrobial protection that meets the highest hygienic standards.

The **BORT select® GenuZip® Knee Support** supports the damaged tissue structures. The support's compressive action supports the postural function of the ligaments thus relieving the joint. At the same time, the support helps prevent incorrect movement.

On the move, the compressive knitting and the silicone pads combine to deliver a massage effect, which reduces the tendency for swelling, helps reduce existing swellings and thus accelerates the healing process. The pad also ensures that the patella is supported during normal movement.


The zip ensures to comfortably taking on and off the support.

USER INSTRUCTIONS


Picture 1:

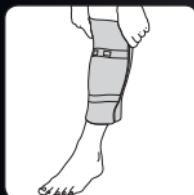The BORT **select[®]** GenuZip[®] has to be closed at the calf by means of the zip.

Picture 2:

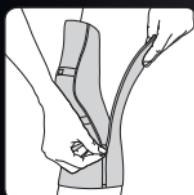Subsequently pull-up closed support and position at the knee.

Picture 3:

For taking off open the zip and remove the support.
After wearing pull into shape, shake out and store it without wrinkles.

INDICATIONS

Chronic post-traumatic or post-operative irritations of knee joint soft tissues, recurrent articular effusion, gonarthrosis, arthritis, i.e. in the case of ligament instability, meniscopathy.

CONTRA INDICATIONS

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

COMPOSITION

58 % polyamide

40 % elastodies

2 % spandex

IMPORTANT NOTE

No reutilisation – this medical device is meant for the care of one patient only.

The **BORT select® GenuZip® Knee Support** supports the healing process particularly during activities. If instructed to wear the support at night, you must take precautions to avoid impairment of blood circulation. If you experience numbness, please loosen the support immediately and, if necessary, remove it.

If you suffer from persistent complaints or if you experience worsening complaints, you should consult with your doctor.

Please do not apply any creams or ointments at any time while wearing the support as these can destroy the material.

The product contains latex and can trigger allergic reactions.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com