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## INSTRUCTIONS FOR USE GB

Dear Customer,

We would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

### FUNCTION

The principal area of application of the SWEDE-O-UNIVERSAL ankle joint support is for general tendon injuries, particularly for the prophylactic treatment of distortions incurred during sports activities.

The inner-shoe design of the SWEDE-O-UNIVERSAL surrounds the ankle joint. The heel remains free. An elongation insert protects the Achilles tendon from direct pressure irritations.

Firmly integrated spiral springs, as well as thermoplastic mouldable plastic **I** reinforcements that can be medially and laterally positioned and feature specific recesses for the internal and external malleolus, provide optimal support and safety during major physical stresses of the ankle joint during sports activities. At the same time the foot retains sufficient flexibility in the plantar flexion and dorsal extension directions.

### INDICATIONS

- Preservative therapy of ankle joint distortions of every severity
- Post-operative further treatment
- Prophylactic treatment of distortions

### CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage)

So far no consequential incompatibilities or allergic reactions have been reported.

### SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

### FITTING

- Thread the laces through all the inner and outer eyelets of the orthosis.
- Pull the SWEDE-O-UNIVERSAL ankle joint support over a sock onto the affected foot.

– The two reinforcements can be inserted together in the corresponding pockets or, depending on the stability requirements, merely on one side of the angle joint **I**.

**Instructions for the orthopaedic technician:**  
The reinforcing inserts should be thermoplastically moulded to the individual anatomy of the malleolus to achieve an optimal fit and stability for the ankle joint.

– Commence by tying the two bottom eyelets firmly, yet comfortable.

**Important for the stability of the ankle joint:**  
Then tie the three subsequent eyelets with sufficient firmness to achieve the best possible stability effect.

– Tighten the laces through the remaining eyelets to produce a comfortable fit.

– Check the distance between the inside lacing strips after the SWEDE-O-UNIVERSAL ankle joint support has been completely applied. The orthosis is too small if the space is larger than 2 cm; the orthosis is too large if the space is less than 0.5 cm. Select the required size.

– If the laces become loose through (sport) activity, then they should be retightened, particularly the three close together eyelets.

### IMPORTANT

The orthotic device should always be worn over stockings. The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

### QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

### CARE

Ensure that the reinforcing inserts are removed from the side pockets before the support is washed.

Wash SWEDE-O-UNIVERSAL in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator.

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